



June 1, 2019

Dear RNA Medical Customer,

After communication from OPTI Medical, in regards to the OPTI CCA instrument, the values for  $pCO_2$  require adjustment. The assigned values for Level 5 of **CVC 123 Calibration Verification Controls**, Lot # 74222 (part of Set Lot #712323), and **QC 823-5 Daily QC for Blood Gas, Electrolyte, Metabolite and BUN**, Lot #74222 are higher than what has recovered in the field. Therefore, it is necessary to revise the mean and range values as outlined below:

Level 5

Lot #: 74222

Exp. Date: 2020-01

Manufacturer/ Analyzer	Current $pCO_2$ Value		Revised $pCO_2$ Value	
	mmHg		mmHg	
	<i>mean</i>	<i>range</i>	<i>mean</i>	<i>range</i>
<b>OPTI Medical</b>				
OPTI CCA	16	14-19	15	11-19

There is no change to the  $pCO_2$  values of levels 1, 2, 3, or 4 of this kit lot or to any other analyte.

We have not reprinted the insert for this lot. This letter will serve as notification of this change. Please keep this announcement with your records for as long as you have this lot in stock.

If you have any questions regarding this revision, please don't hesitate to call or email me using the contact information provided.

Sincerely,

Rachel Casazza  
Product Manager  
RNA Medical: 1-978-772-9070  
[rachel.casazza@bio-techne.com](mailto:rachel.casazza@bio-techne.com)

BIOMKT-210  
Rev. B