ENGLISH

INTENDED USE

RNA Medical® Safe-Wrap Combo Blood Collection Tubes are used for the collection of capillary blood samples for use with cartridge-based blood analysis systems. It may be used with the Abbott Point of Care i-STAT® System 95 µL cartridges (G3+, CG4+, EG6+, EG7+, CG8+, and CHEM8+) and 65 µL cartridges (6+, EC8+, EC4+, E3+, G, and Crea). This product is not for use with either coagulation or immunoassay cartridges and should not be used to transfer aqueous control samples.

PRODUCT DESCRIPTION

Safe-Wrap Combo Blood Collection Tubes are Mylar®-wrapped glass capillary blood collection devices that have been treated with calcium-balanced lithium heparin to prevent the sample from clotting. The Mylar wrapping serves to minimize the risks associated with broken glass capillaries, including glass cuts and aerosol formation, by containing both the glass and the sample in the event of accidental breakage. This product is packaged in cylinders containing 50 tubes and 50 dispensing plungers.

Each Safe-Wrap Combo Blood Collection Tube has one (1) red and two (2) black bands. The broad band closest to the red band represents the 95 µL fill line and the narrow band furthest from the red band represents the 65 µL fill line. A porous plug is positioned at the 95 µL fill mark (see Figure 1).

Active Ingredients:

Safe-Wrap Combo Blood Collection Tubes are treated with calcium-balanced lithium heparin (derived from porcine intestinal mucosa) at a concentration of approximately 70 IU/mL of whole blood sample.

Caution: Practice the recommended "Universal Precautions" when handling and using this device. Discard as biohazardous waste after use.

STORAGE

The expiration date stated on the Safe-Wrap Combo Blood Collection Tubes packaging is for product stored at temperatures of 2-30 °C.

DIRECTIONS FOR USE

Follow the policies and procedures for capillary blood collection documented by your facility.

1. Remove one capillary tube and one plunger. Do not touch the tip furthest from the red band.
2. Insert the plunger in the end with the red band as shown in figure 1 below. If a 65 µL sample is desired, use the plunger to reposition the porous plug to the 65 µL fill mark. The porous plug will stop the blood at the fill mark at which it is positioned.
3. Hold the capillary tube so the tip furthest from the red band is pointed toward the puncture site and so gravity can aid the flow of blood into the capillary tube.
4. Touch the tip to the blood drop and allow the blood to fill to the porous plug.
5. Introduce the sample to the cartridge (as directed by the cartridge manufacturer) by depressing the plunger until the entire sample is dispensed.
6. Discard the capillary tube and plunger as biohazardous waste.

Consider the following for sample collection and handling:

- For blood gas sample collection, the puncture site should be warmed to increase blood flow.
- Allow alcohol applied to puncture site to dry before performing the puncture.
- Wipe away the first drop of blood to avoid contamination with tissue fluid.
- Collect blood from the center of the blood drop after it forms.
- Avoid squeezing the puncture site excessively during collection as hemolysis may occur.
- Do not use the capillary tube if an air bubble has been trapped in the sample. This may result in a quality check code (with the i-STAT System) or erroneous pO₂ results.
- Do not use the sample if it clots or if the glass breaks.

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Safe-Wrap is a trademark of Bionostics, Inc., Devens, MA, USA. RNA Medical is a registered trademark of Bionostics, Inc., Devens, MA, USA. Mylar is a registered trademark of E. I. DuPont, Inc., Wilmington, DE, USA. i-STAT is a registered trademark of Abbott Laboratories, Abbott Park, IL, USA.
RNA Medical, Division of Bionostics, Inc.