## CC 525 Blood Gas•Electrolyte•Metabolite•CO-Oximeter (Bayer) Control Level 1

ESPAÑOL

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RNA Medical is a registered trademark of Bionostics, Inc. The products described herein are covered by one or more of the following U.S. Patents and their foreign counterparts: 5,558,985; 5,320,965; 5,304,491; 5,045,529; 5,013,666; 4,945,062; 4,843,013; 4,753,888.

# CC 525 Level 1

## Blood Gas•Electrolyte•Metabolite•CO-Oximeter (Bayer) Control

<b>LOT</b>	73613	<b>REF</b>	CC 525-1
<b>Exp.:</b>	2010-03	<b>CE</b>	<b>IVD</b> For In Vitro Diagnostic Use

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Before use, hold the ampul at the top and bottom (with forefinger and thumb) and shake for 10 seconds to mix the solution. Tap the ampul to restore the liquid to the bottom. Use gauze, tissue, gloves, or an appropriate ampul opener to protect fingers from cuts and open the ampul by snapping off the tip at the score. Immediately introduce the liquid from the ampul to the analyzer, following the instrument manufacturer's instructions for sampling a control material. Use direct aspiration, syringe transfer, or capillary mode techniques.

For pH/blood gas values, the control should be analyzed immediately after opening. For electrolyte, glucose, lactate, and CO-Oximeter measurements, CC 525 is stable for up to one (1) hour after opening.

For pH/blood gas values, the control should be analyzed immediately after opening. For electrolyte, glucose, lactate, and CO-Oximeter measurements, CC 525 is stable for up to one (1) hour after opening.

Before use, hold the ampul at the top and bottom (with forefinger and thumb) and shake for 10 seconds to mix the solution. Tap the ampul to restore the liquid to the bottom. Use gauze, tissue, gloves, or an appropriate ampul opener to protect fingers from cuts and open the ampul by snapping off the tip at the score. Immediately introduce the liquid from the ampul to the analyzer, following the instrument manufacturer's instructions for sampling a control material. Use direct aspiration, syringe transfer, or capillary mode techniques.

The values for each control analyte on the enclosed Expected Values Chart are based on multiple determinations performed on randomly selected samples from each lot. The listing for each instrument represents the expected range and mean value of this range for ampuls that are at 25 °C when tested. (Note: pO<sub>2</sub> values will vary inversely by about one percent (1%) per degree Celsius that the temperature of the ampul varies from 25 °C.)

The Expected Values are provided as a guide in evaluating analyzer performance. Since instrument design and operating conditions may vary, each laboratory should establish its own expected values and control limits. The mean value established should fall within the Expected Value range shown on the chart.

RNA Medical provides monthly statistical reports for tracking and review of analyzer performance as well as lot number specific peer group data. Please contact RNA Medical for information about this service.

### LIMITATIONS

- CC 525 is sensitive to many instrument related factors that would affect analytical results. Because it is not a blood-based material, it may not detect certain malfunctions that would affect the testing of blood.
- This product is intended for use as a quality control material and can assist in evaluating the performance of laboratory instruments. It is not for use as a calibration standard and its use should not replace other aspects of a complete quality control program.
- Dye-based controls may color the protein that builds up on system components. Studies show that this coloration does not affect system performance. To remove protein build-up, perform maintenance procedures recommended by the instrument manufacturer.

RNA Medical is a registered trademark of Bionostics, Inc. The products described herein are covered by one or more of the following U.S. Patents and their foreign counterparts: 5,558,985; 5,320,965; 5,304,491; 5,045,529; 5,013,666; 4,945,062; 4,843,013; 4,753,888.