

November 9, 2009

Dear RNA Medical Customer,

A review of RNA Medical CC527 Blood Gas, Electrolyte, CO-Oximeter (ITC AVOXimeter) Control, Level 2, Catalog # CC 527-2, lot number 93416 and Catalog # CC 527, set lot number 9333666, revealed the need for revised O₂Hb Mean and Range values for the AVOXimeter 4000. These revisions are required to address a misprint in the level 2 insert. Revised values for the O₂Hb Mean and Range for the AVOXimeter 4000 are provided below.


AVOXimeter 4000 %O₂Hb

Level	Lot Number	Exp. Date	Instrument	Current Mean	Current Range	Revised Mean	Revised Range
2	93416	2012-05	AVOXimeter 4000	81.6	82.9 – 80.2	86.6	82.9 – 90.2

RNA Medical has not reprinted the inserts for this lot, but has included an additional insert card in the affected boxes in our stock. This letter, along with the additional insert card, will serve as notification of this change to the customers whom have received the product prior to this review. Please keep this announcement with your records as long as you have these lots in stock.

Please do not hesitate to contact RNA Medical by phone at 1-800-533-6162 or email at technicalsupport@RNAMedical.com if you should have any questions regarding this revision.

Sincerely,

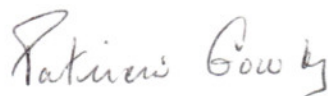

Patricia Gowdy
Marketing and Technical Support
RNA Medical

BIOMKT-144
Rev. A