



July 7, 2011

Dear RNA Medical Customer,

A review of RNA Medical CVC 123, Calibration Verification Controls for Blood Gas Analyzers, **set lot 012318, expiration date of 2013-07**, has prompted the assigned values for Siemens 1200 Analyzers to be revised. The revisions apply only to: level one pO_2 , Cl^- , and Na^+ , level 2 pO_2 and level 5 Ca^{++} .

See chart below.

**RNA Medical CVC123 Set Lot 012318, Expires 2013-07
Revised Target Values for Siemens 1200 Series Instruments**

Level	Lot	Analyte	Current Mean	Current Range	Revised Mean	Revised Range
1	04115	pO_2	37	22-52	34	19-49
1	04115	Cl^-	66	61-71	71	66-76
1	04115	Na^+	80	75-85	84	79-89
2	01521	pO_2	71	61-81	68	58-78
5	04215	Ca^{++}	0.30	.20-.40	ORL ¹	N/A ²

1. Outside (Analyzer's) Reportable Limits
2. Non Applicable

RNA Medical has not reprinted the inserts for this lot, so please use this letter as notification for these changes. Please keep this announcement with your records as long as you have these lots in stock. This letter will also be available on our website at: www.RNAMedical.com

Please do not hesitate to contact RNA Medical by phone at 1-800-533-6162 or by email at: technicalsupport@rnamedical.com if you should have any questions regarding this revision.

Sincerely,

Patricia Gowdy
Marketing and Technical Support
RNA Medical

BIOMKT-150
Rev. A 7/2011